# X-TRIM 4

855.12

CE

Regulation (EU)  
2017/745

EN

[WWW.KONG.IT](http://WWW.KONG.IT)

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# TABLE OF CONTENTS

	<b>1 - SYMBOLS AND SUPPORT</b>	<b>4</b>
	1.1 Symbols	4
	1.2 Support	4
	<b>2 - GENERAL INFORMATION</b>	<b>5</b>
	<b>3 - TECHNICAL FEATURES</b>	<b>6</b>
	3.1 Parts terminology and materials	6
	3.2 Dimensions	7
	3.3 Capacity	7
	3.4 Optional components and spare parts	7
	<b>4 - SPECIFIC INFORMATION</b>	<b>8</b>
	4.1 Applications	8
	4.2 Assembly	8
	4.3 Mounting the “HRP” Head immobiliser	10
	4.4 Immobilising the patient	11
	4.5 Unlocking the patient	14
	<b>5 - MAINTENANCE AND REPAIR</b>	<b>15</b>
	5.1 General Information	15
	5.2 Maintenance	15
	5.3 Repair	15
	<b>6 - STORAGE</b>	<b>16</b>
	<b>7 - CHECKS, INSPECTIONS AND REVISIONS</b>	<b>17</b>
	7.1 Checks	17
	7.2 Inspections	17
	7.3 Revisions	17
	<b>8 - DURATION AND WARRANTY</b>	<b>18</b>
	8.1 Duration	18
	8.2 Disposal	18
	8.3 Warranty	18
	8.4 Legal requirements	18
	<b>9 - LABELLING AND SYMBOLS</b>	<b>19</b>
	9.1 Labelling	19
	9.2 Symbols	19
	<b>10 - DOCUMENTS</b>	<b>20</b>
	10.1 Inspections and revisions register	20
	10.2 Maintenance and repair register	21
	10.3 Declaration of conformity UE	22
	<b>11 - LIST OF APPLIED AND REFERENCE STANDARDS</b>	<b>23</b>
	11.1 Applied standards	23
	11.2 Standards used as reference	24

### 1.1 SYMBOLS

To make reading this manual comfortable and clear, the following are the symbols used to manage important warnings, for correct and safe use of the device.

#### **REQUISITE FOR CORRECT USE**

It identifies the presence of information required to use the device correctly.

---

#### **INFORMATIVE REQUISITE**

It identifies the presence of useful and general information, to guide the user to consciously use the device and/or carry out the actions.

---

It identifies that the Medical Device is manufactured, designed and produced in compliance with the provisions of the General Safety and Performance Requirements of Regulation (EU) 2017/745 (Class I medical device, in compliance with classification rule 1 as indicated in the attachment VIII).

---

### 1.2 SUPPORT

For information, contact Kong Customer Service by:

- telephone +39 0341 630506,
- fax +39 0341 641550,
- email: [safetycare@kong.it](mailto:safetycare@kong.it),

or write to KONG S.p.A. – Via XXV Aprile, 4 – 23804 Monte Marenzo LC - ITALY.

To facilitate assistance operations, always communicate or indicate the serial number (SN) indicated on the label applied to the Medical Device.

# CHAPTER 2

## 2 GENERAL INFORMATION

The information provided by the manufacturer (hereinafter referred to as information) must be read and clearly understood by the user prior to using the Medical Device (MD hereafter). The information regards the description of the features, performance, assembly, disassembly, maintenance, preservation, disinfection, etc. of the device. Even though the information offers tips on use, this information shall not be considered as a user manual under actual conditions of use.

### WARNINGS AND LIMITATIONS OF USE:

- this device shall be strictly used by people who are physically fit, trained (instructed and taught) to use the device and with specific experience regarding moving the patient or, during training activities, by people under direct supervision of the trainers/ supervisors who guarantee the safety thereof,
- do not use the device before fully reading and understanding this user manual,
- strictly follow the manufacturer's information, improper use of the device is hazardous,
- modifying and/or repairing the device is strictly forbidden,
- all checks described in chapter 7 must be carried out prior to and after using the device. In case of any doubt on the efficiency of the device, the user must replace it immediately,
- non-compliant use, deformations, falls, wear, chemical contamination, exposing textile/plastic components/devices to temperature below  $-30^{\circ}\text{C}$  or above  $+50^{\circ}\text{C}$  and metal components/devices to temperatures

- exceeding  $100^{\circ}\text{C}$  are some examples of causes that can reduce, limit and end the life of the device,
- Prior to any rescue operation, be keen not to exceed the capacity indicated in paragraph 3.3,
- in order to reduce risks of exposure to / transmission of infectious diseases, clean and disinfect the device as indicated in chapter 5,
- improper use of the patient immobilisation systems can jeopardise the safety of the patient,
- always check the compatibility of the devices used alongside the device by consulting the relative manufacturer's information,
- use of spare parts or optional components different from the ones indicated in paragraph 3.4 can be hazardous,
- do not expose the device to sources of heat and at contact with chemical substances. Reduce exposure to direct sunlight as much as possible. At low temperatures and in the presence of humidity, formation of ice could reduce flexibility and increase cutting and abrasion-related risks on textile and synthetic devices.
- Report to the manufacturer and the competent authority of the member state where the user and / or patient is established any serious incident occurring in relation to the MD.

All our devices are tested/ checked piece by piece in compliance with the procedures laid down by the Quality System certified in accordance with the UNI EN ISO 9001 standard. Laboratory tests, testing, information and standards do not always reproduce the practical result. Thus, the results obtained under the actual conditions of use of the device in the natural environment may differ, even considerably at times. The best information lies in the continuous practical use under the supervision of skilled/expert/qualified people.

**Warning: not suitable for use in an ATEX environment (Directive 94/9/CE)**

# CHAPTER 3

## TECHNICAL FEATURES

### 3.1 PARTS TERMINOLOGY AND MATERIALS

- A – Carbon fibre spinal board
- B – Carbon fibre spars
- C – Nylon hinge
- D – Nylon rivets
- E – Brass rear hinges
- F – Carbon fibre crossbars
- G – Taylan Belt

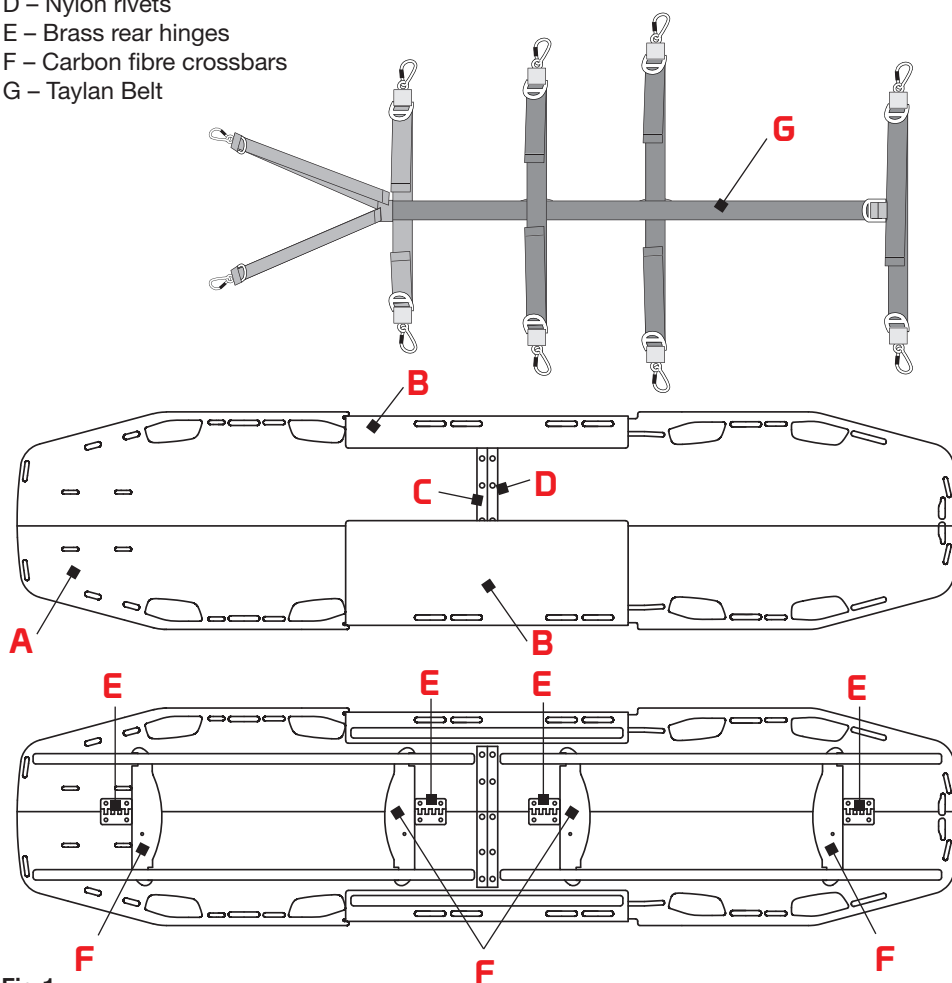
Fig.1

## 3.2 DIMENSIONS

Open: 182x42x4 cm

Folded: 91x24x8 cm

Weight: 6,8 kg

## 3.3 CAPACITY

The load capacity of the “X-TRIM 4” spinal board is defined according to the results of the laboratory tests and the recommended safety coefficients.

How to use	Capacity
Manual handling	150 kg

Prior to any rescue operation, ensure not to exceed the specified capacity!

## 3.4 OPTIONAL COMPONENTS AND SPARE PARTS

The methods of connection and use of the optional components are defined in the relative user instructions.

### 3.4.1 Optional components

- 855.06 HRP: head immobilisation device

### 3.4.2 Spare parts

C - Nylon hinge.

D - Rivets for fixing the nylon hinge.

G - Taylan Belt.

Rivet breakage is provided to prevent damage to the spinal board should it be bent leaving material inside.

# CHAPTER 4

## SPECIFIC INFORMATION

### 4.1 APPLICATION

The “X-TRIM 4” spinal board is a medical device for patient immobilisation suitable for any type of rescue, including rough terrain. The minimal weight and thickness facilitates roll-on manoeuvres. Decisions concerning moving and immobilising the patient, as well as the duration, method to be used and combination with other devices, shall be taken and executed by expert and trained personnel only.

### 4.2 ASSEMBLY

a) Open the spinal board (Fig. 2),

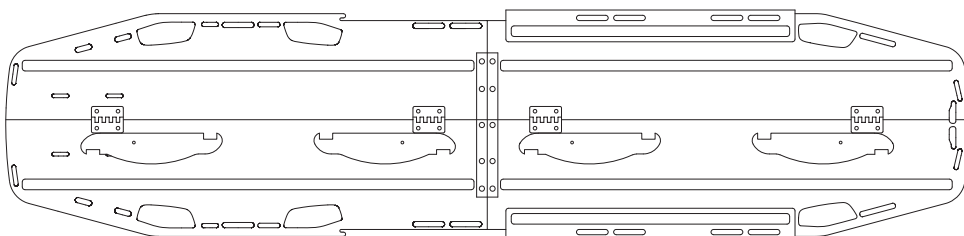
Fig.2

b) rotate the crossbars (F) until they lock into their slots (fig. 3),

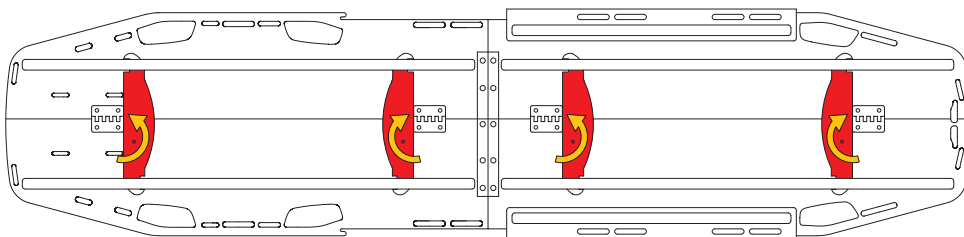
Fig.3

c) Slide the stiffening spars until they lock into the lateral grooves of the spinal board (fig. 4),

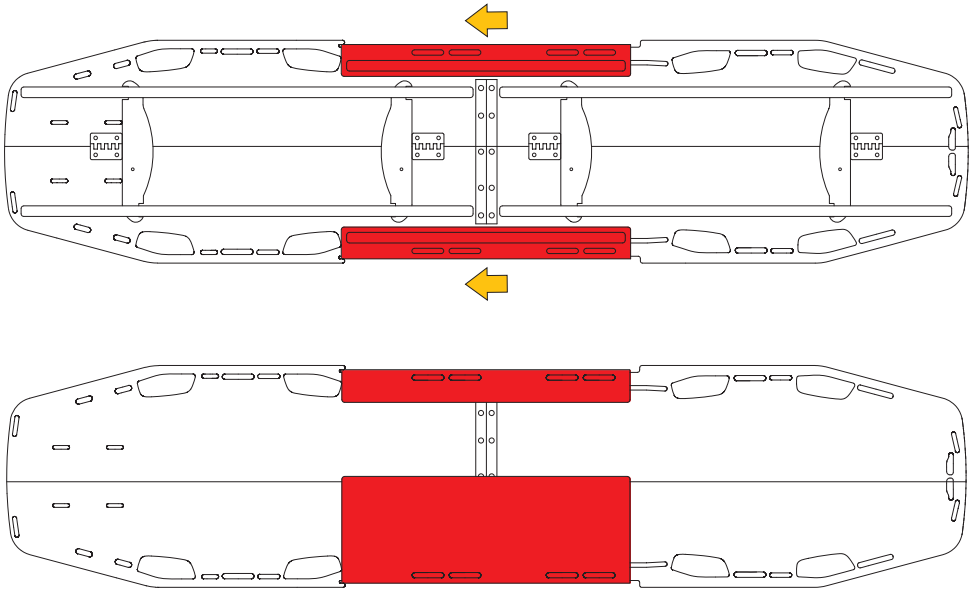
Fig.4

## 4.3 MOUNTING THE HRP HEAD IMMOBILISER

a) Insert the Velcro straps into the slots of the head immobiliser and spinal board (Fig. 5).

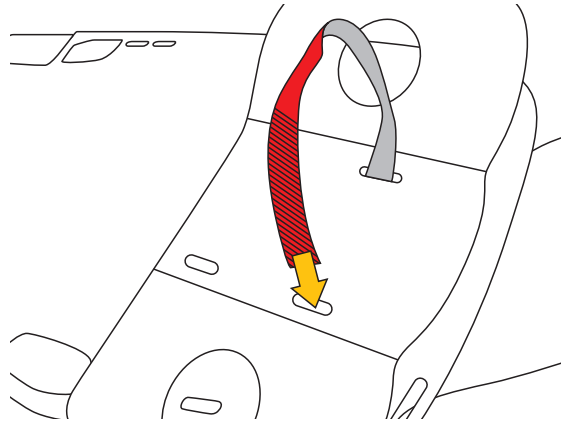
Fig.5

b) Attach the head immobiliser by pulling and overlapping the Velcro straps on the underside of the spinal board (fig. 6).

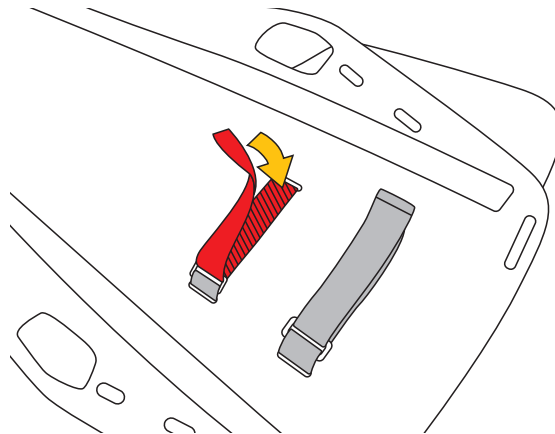
Fig.6

## 4.4 IMMOBILISING THE PATIENT

With the spinal board positioned on a flat and stable surface:

- a) place the patient's head in the head immobiliser and immobilise it using the padded straps (Figs. 7 and 8),

The head immobiliser is also suitable for immobilising patients with cervical collars.

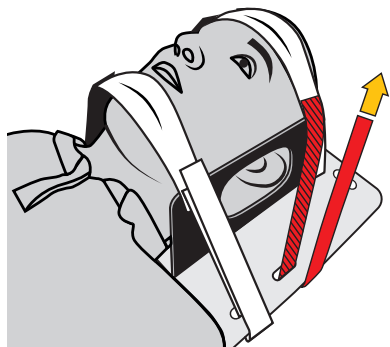
Fig.7

- b) Position the “TAYLAN” belt as shown in figure 8,

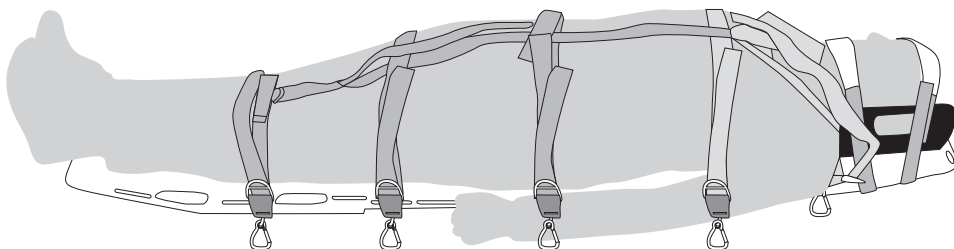
Fig.8

c) fasten the red straps (shoulders and torso) by hooking the connectors to the slots on the board, as shown in Figures 9 and 10, and tighten them slightly by using the buckles (Figures 11 and 12),

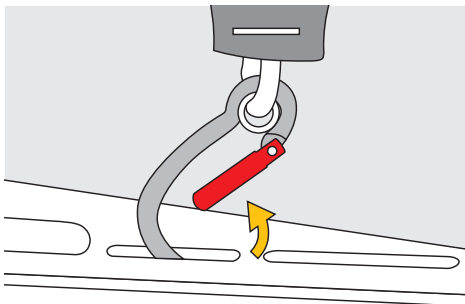
Fig.9

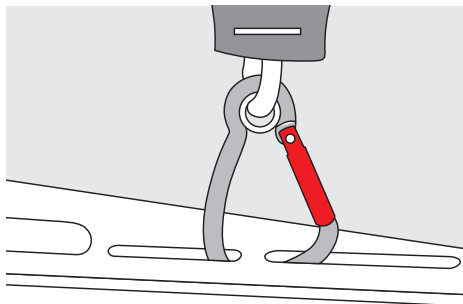
Fig.10

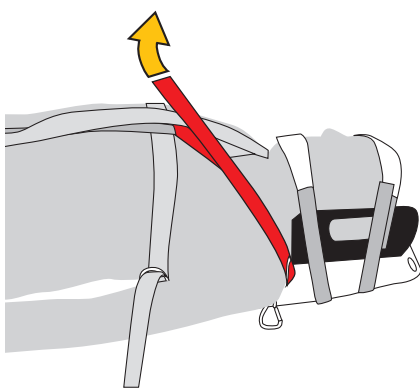
Fig.11

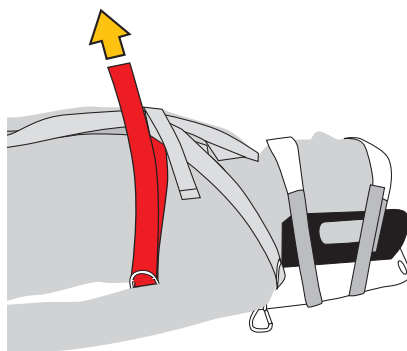
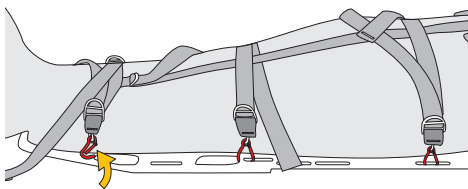
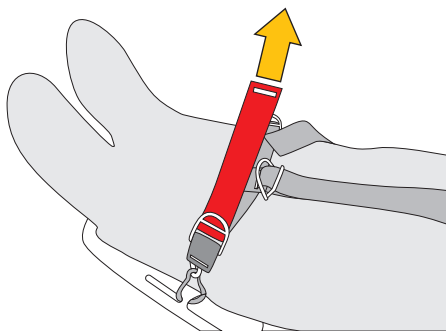
Fig.12

d) fasten the black straps (pelvis and legs) by hooking the connectors to the slots on the board and slightly tighten them using the buckles (figs. 13 and 14)

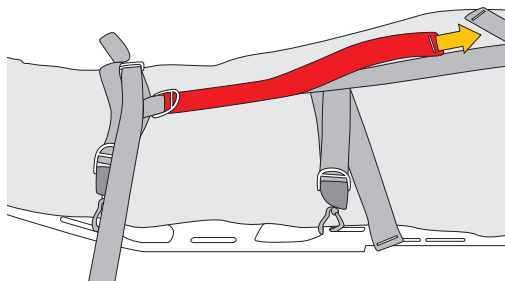
**Fig.13**

**Fig.14**

e) conclude the immobilisation by using the longitudinal lock (fig. 15),

**Fig.15**

f) the patient fully immobilised on the spinal board can be moved by the operators.  
Avoid prolonged contact with the patient's skin with the device's materials.

## 4.5 UNLOCKING THE PATIENT

a) Loosen the straps by using the buckles as shown in figure 16.

If the straps are removed from their buckles, they can be reinserted as shown in figures 17-18-19,

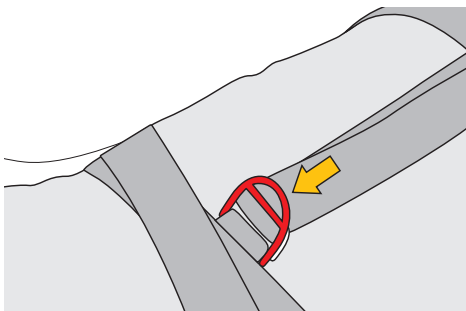
Fig.16

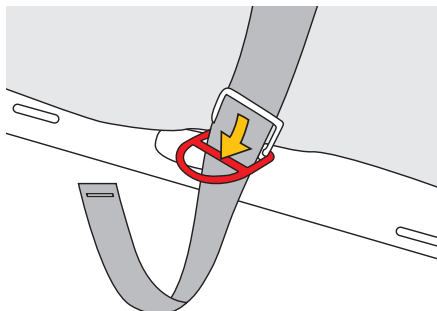
Fig.17

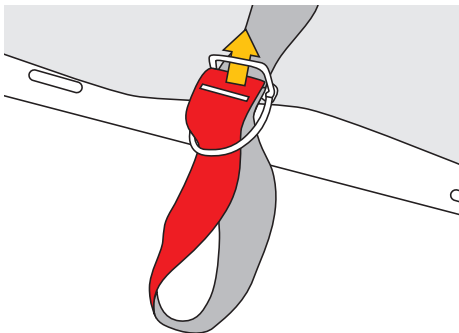
Fig.18

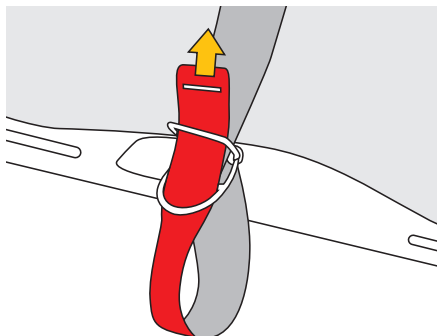
Fig.19

# CHAPTER 5

## MAINTENANCE AND REPAIR

### 5.1 GENERAL

The MD spinal board is made of materials that are highly resistant to wear and to external agents. Despite this, the use conditions require maintenance operation and even repair in particular conditions.

Maintenance and repair operation must be recorded in a special register an example of which is indicated in chapter 10.

### 5.2 MAINTENANCE

The maintenance operations that must be carried out by the user are, in order:

- a) cleaning: after every use, wash with warm (max. 40 C) potable water, possibly adding a neutral detergent (e.g. Marseille soap). Rinse and leave to dry in a shaded place, away from direct sources of heat,
- b) disinfection, when deemed necessary or after prolonged inactivity (more than 21 days), preceded by cleaning: wipe with a clean cloth , soaked in a water solution of sodium hypochlorite (bleach) with a 0,1% concentration (1000 ppm). When blood or other bodily fluids are present, the recommended concentration of sodium hypochlorite is 0,5%(5000 ppm)

### 5.3 REPAIR

Repair shall be strictly carried out by the manufacturer.

The user is allowed to replace the parts mentioned in paragraph 3.4.2 only, with new and original parts.

# CHAPTER 6

## STORAGE

### 6 STORAGE

After cleaning, any disinfection and drying operations, keep the product and its optional components in a place that is dry (relative humidity 40-90%), cool (temperature 5-40° C) and safe (avoid UV radiations), chemically neutral (absolutely avoid salty and/or acid environments), away from sharp edges, sources of heat, humidity, corrosive substances or other possible jeopardising conditions.

**Do not store this device wet!**

### 7.1 CHECKS

Before and after every use, check the device and ensure that:

- it is appropriate for the use it was intended for,
- it has not been subjected to mechanical deformations and it neither has cracks nor shows signs of wear,
- the textile parts do not have cuts, burns, chemical products residues, excessive hair, wear, in particular check areas at contact with the metal components,
- markings, including labels, are readable.

### 7.2 INSPECTIONS

The device must be inspected at least once a year, starting from the date of first use, by expert personnel trained and approved by KONG S.p.A. The date of the first use and the outcome of the inspection must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

### 7.3 SERVICING

The device must be serviced by KONG S.p.A., or by expert personnel trained and approved by KONG S.p.A. if:

- any malfunctions are detected,
- the result of the pre and post use check or of the inspections is negative.

The outcome of the servicing must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

The device subject of servicing has a one-year warranty from the date of servicing.

# CHAPTER 8

## PRODUCT LIFE AND GUARANTEE

### 8.1 PRODUCT LIFE

Carefully read the “Warnings and limitations of use” paragraph in chapter 2. The duration of this device depends on the outcome of the at least one-yearly inspection (paragraph 7.2) and possible servicing.

**Important: any devices that have not passed the pre-use, post-use and periodic inspection or servicing must be eliminated and made unusable.**

### 8.2 DISPOSAL

Follow the rules in force in the country of use and the waste disposal procedures of the reference hospital facility for proper disposal.

### 8.3 GUARANTEE

The manufacturer guarantees the compliance of the device with the regulations in force at the time of production. Warranty covering defects is limited to defects regarding raw materials and manufacturing defects. The warranty does not cover normal wear, oxidation, damage caused by non-compliant use and/or in competition, by improper maintenance, transportation, preservation or storage. Warranty will be immediately deemed null and void should the device be modified or tampered with. Validity corresponds to the legal validity of the country where the device was sold, as from the date of sale, by the manufacturer. No claims shall be raised against the manufacturer once the aforementioned period has expired. Any repair or replacement request under warranty must be accompanied by a proof of purchase. Should the defect be recognised, the manufacturer undertakes to repair or, replace or refund the device at his discretion. The manufacturer shall not be held liable beyond the invoice price of the device in any case whatsoever.


### 8.4 LAW OBLIGATIONS

Professional and free time activities are often regulated by special national laws which can impose limits and/or obligations concerning the use of these devices. The user has the obligation to know and apply such laws which could provide for limits different from those indicated in this information.

# CHAPTER 9

## LABELLING AND SYMBOLS

### 9.1 MEDICAL DEVICE LABEL

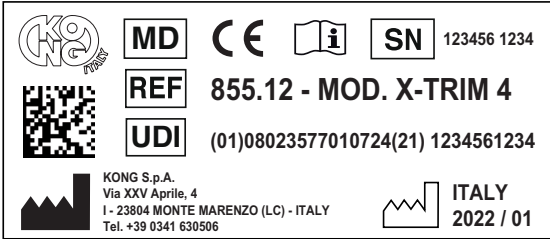Any changes to the positioning of the symbols does not change the content thereof.


### 9.2 SEPARABLE COMPONENTS LABEL

Any changes to the positioning of the symbols does not change the content thereof.

### 9.3 SYMBOLS

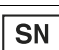#### Identification and traceability

	Medical Device
	Product identification code
	Unique Device Identifier
	Manufacturer's identification
	Unique serial number
	Country, year and month of production.

Data matrix UDI  
 Readable with the Apps:  
 iGepir - BarValid - Barcode Scanner

#### Information and safety

	Consult the user manual
	Compliant with Regulation (EU) 2017/745

# CHAPTER 10

## DOCUMENTS

### 10.1 INSPECTIONS AND SERVICING REGISTER

SPINAL BOARD - Class I MD				
REF.	855.12	MOD.	X-TRIM 4	
UDI-DI	08023577010724	SN		
Date of first use				
INSPECTIONS AND SERVICING				
DATE	I/S	Description	Outcome	Supervisor

## 10.2 MAINTENANCE AND REPAIR REGISTER

<b>SPINAL BOARD - Class I MD</b>				
REF.	855.12	MOD.	X-TRIM 4	
UDI-DI	08023577010724	SN		
Data primo utilizzo				
<b>MAINTENANCE AND REPAIRS</b>				
DATE	M/R	Description	Outcome	Supervisor

# 10.3 EU DECLARATION OF CONFORMITY

The EU Declaration of Conformity is available for download at [www.kong.it](http://www.kong.it)

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Declares under its sole responsibility that the device:

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V080504 – Patient transfer boards and tables

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Spinal board for transporting patients during rescue operations

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[www.kong.it](http://www.kong.it)

## 11.1 APPLIED STANDARDS

- EN 597-1: 2015  
Furniture - Assessment of the ignitability of mattresses and upholstered bed bases - Part 1: Ignition source smouldering cigarette
- EN 1865-1: 2010 + A1: 2015  
Patient handling equipment used in road ambulances - Part 1: General stretcher systems and patient handling equipment
- EN 62366-1: 2015 + A1: 2020  
Medical devices - Application of usability engineering to medical devices
- EN ISO 846:2019  
Plastics - Evaluation of the action of microorganisms
- EN ISO 10993-1: 2020  
Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
- EN ISO 10993-5: 2009  
Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-23: 2021  
Biological evaluation of medical devices - Part 23: Tests for irritation
- EN ISO 14971: 2019  
Medical devices - Application of risk management to medical devices
- EN ISO 15223-1: 2021  
Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements
- EN ISO 20417: 2021  
Medical devices - Information to be supplied by the manufacturer.

## 11.2 STANDARDS USED AS REFERENCE

- EN ISO 13485: 2016  
Medical devices - Quality management systems - Requirements for regulatory purposes
- EN 13718-1: 2014 + A1: 2020  
Medical vehicles and their equipment - Air ambulances - Part 1: Requirements for medical devices used in air ambulances.

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